



The FAWSSit™

Fold Away Wheelchair Shower

QUICK START GUIDE

US Patent No., 7,231,673 B1



Step 1:
Stand Unit Upright



Step 2:
Pull right side open



Step 3:
Pull left side open



Step 4:
Lower dark blue water pan



Step 5:
Roll backward into shower



Step 6:
Lower front safety bar



Step 7:
Connect water pan to shower
with attached velcro strips




Step 8:
Front shower curtain can be
closed for privacy



Step 9:
Attach showerhead
to faucet



Step 10:
Place water waste
tube in drain

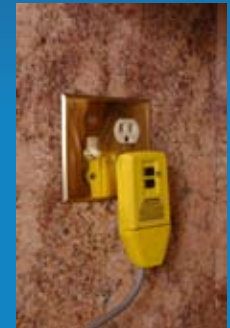

Step 11:
Connect power cord
to 110v outlet



Step 12:
Turn on the pump

Congratulations!